Excilor®

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Excilor®

Skin Cracks Hands - Feet SKIN CRACKS AND FISSURES

PROTECTIVE FILM BARRIER EFFECT

REDUCES PAIN, HELPS HEALING

For small fissures and skin cracks of hands and feet.

Excilor® Skin Cracks Hands - Feet is indicated for the treatment of small fissures, chapped skin, cracked hands and feet, and small cuts.

PROPRIETIES

Thanks to its particular technology, once applied, Excilor® Skin Cracks Hands - Feet is transformed into an elastic, transparent, protective and water resistant film, that dries rapidly. The film protects the fissure from external aggressive agents, reduces pain, helps heal skin cracks and prevents lesions from reopening.

Excilor® Skin Cracks Hands - Feet has a special formula that is enriched with:

- Vitamin E, a pre-eminent antioxidant that favours skin regeneration
- Castor oil, an excellent emollient and elasticising agent of the protective layer

It is also indicated for the treatment of small fissures in diabetic feet as no contraindications exist.

INSTRUCTIONS FOR USE

- Clean and dry the lesion, until it stops bleeding
- Apply the product using the applicator, completely covering the lesion to be treated
- Let it dry for a few seconds
- Repeat the application 2-3 times a day until the lesion has completely healed
- To remove the film, just apply a drop of the product on the pre-existing filmogel
- Close the bottle carefully after use
- After opening, the product must be used within 6 months

WARNINGS

- Do not use on children under 6 years of age
- Keep out of the reach of children
- Use on localised, small-sized lesions
- After applying, a slight burning sensation may be briefly felt, which disappears after a few seconds
- If reddening occurs, stop using the product and consult your doctor
- Do not swallow
- Do not use after the expiry date printed on the pack
- Do not use if the package has been opened or is damaged
- Store in a cool dry place away from heat sources
- Highly flammable
- Irritating for eyes

CONTRAINDICATIONS AND SIDE EFFECTS

Do not apply on:

- Bleeding lesions
- Infected lesions
- Burns
- The mucosa (lips, anus)
- Breasts
- The area around the eyes
- Do not use if you have known sensitivity to any of the components

COMPOSITION:

ALCOHOL DENAT, CASTOR OIL, ETHYL ACETATE, HYDROXYPROPYL CELLULOSE, ISOPROPYL ALCOHOL, NITROCELLULOSE, TOCOPHERYL ACETATE.

PACK

7ml bottle with applicator.

80 applications

Safety cap

MEDICAL DEVICE  0546

 Pasquali S.r.l.
V.le Belfiore 41 - 50144 Firenze - ITALY

Rev.08 - January 2017

Do not dispose of in the environment

Warning!
Read carefully the package leaflet

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Vemedica Pharma Hispania S.A.
Vemedica Pharma Portugal S.A.
Vemedica B.V.